

# ActiveCare Medical

## AT1000 Scooter Trailer



### Contents

1. Introduction
2. Parts
3. Assembly
4. Specifications
5. Warnings

# 1. Introduction

Your **AT1000 Scooter Trailer** provides an easy and convenient way to carry items with you when using your **ActiveCare Medical** scooter. It's perfect for carrying whatever you need for your favorite outdoor activities. The AT1000 attaches easily to the rear of your Prowler™ or Osprey™ scooter and can be folded quickly for storage or transport.

# 2. Parts



- 1. Connection Bar
- 2. Basket Cover
- 3. Frame
- 4. Wheels (quick release)
- 5. Front Legs
- 6. Fabric basket

### 3. Assembly

1. Place the fabric basket over the top of the trailer frame so that the top flap of the fabric cover overlaps the top of the frame. Use the snaps in the fabric basket's top flap to fasten the basket into place.



#### Note

The fabric basket features an attached cover that can be snapped into place to protect items placed in the trailer. The cover can be rolled back out of the way when not needed.



2. After attaching the fabric basket to the trailer frame, locate your scooter's accessory connection point. This is a square metal tube located at the back of your scooter's seat.



Insert the trailer's connection bar into the accessory connection point and then tighten the screw to lock the connection bar in place.



Note

Because of the difference in height between the Prowler and Osprey accessory connection points, the connection bar must be turned the proper way to correctly attach the trailer to your scooter. See the images below for the proper connection bar positioning:



**Prowler 3410/3310**



**Osprey 4410**

3. The trailer's wheels are removable. To remove the wheels, loosen the screw located on the inside of each wheel, the lift up on the hinged piece on the frame to release the wheel.



To reattach the wheels, insert the wheel's metal pin into the appropriate hole in the frame while holding up the hinged piece. Once the wheel is inserted, release the hinged piece and tighten the screw to lock the wheel back in place.



4. Your trailer can be folded for easy storage and transport. Simply push the front legs back toward the wheels.



#### 4. Specifications

Overall Dimensions	25" (L) x 25" (W) x 22" (H)
Tires	12" PU tire
Total Weight	21.25 lbs.
Weight capacity	55 lbs.
Compatible Scooters	Prowler 3310 Prowler 3410 Osprey 4410
Color	Gray

## 5. Warnings

1. When using the trailer is attached to your scooter, **DO NOT** swivel the scooter's seat.
2. When using the trailer with your scooter, the maximum safe incline to drive on is 6 degrees.
3. When driving your scooter in reverse, or turning, when the trailer is attached, please pay careful attention to whether any people or obstacles are nearby. When traveling in reverse, the trailer will angle in the direction **opposite** of your scooter's direction.
4. For safety, **DO NOT** carry people or animals in your trailer.